

## BENEFITS



### HEALTHY EYES

Fewer Deposits, Less Irritation  
= Less Discomfort



### SURFACE PROPERTIES

Distinctly wettable  
and very lubricious



### BETTER WEARING EXPERIENCE

Patients prefer their custom  
lenses coated in Tangible Hydra-PEG™



### JUST HOW COMFORTABLE?

Tell us how comfortable  
your new lenses are

## CONGRATULATIONS!

Your custom contact lens is coated with  
Tangible Hydra-PEG™ technology to enhance  
contact lens wettability, lubricity, and surface quality.

To maximize the life of your Tangible Hydra-PEG™  
coating, it is very important to comply with the  
following Lens Care and Handling Instructions.

## LENS CARE & HANDLING INSTRUCTIONS



**HANDLE WITH CAUTION,  
LENSES ARE SLIPPERY**

Plug your drain, to avoid  
losing your lenses



**DO NOT USE TAP WATER**

Use buffered, preservative-free  
sterile saline solution to  
rinse your lenses



**DO NOT USE ENZYMATIC,  
ABRASIVE OR ALCOHOL  
BASED CLEANERS**

They may damage the  
Tangible Hydra-PEG™ coating  
and should not be used



**USE TANGIBLE HYDRA-PEG™  
APPROVED CLEANERS**

Optimum by Lobob®  
CDS Cleaning/Disinfecting/Storage Solution  
by Lobob Laboratories  
or  
Menicon Unique pH®  
Multipurpose Solution by Menicon  
or  
Clear Care®  
Cleaning & Disinfection Solution by Alcon

PATIENT'S NAME

EYE CARE PROFESSIONAL

©Contamac