

SOLUTION COMPATIBILITY WITH TANGIBLE HYDRA-PEGTM (U.S. MARKET)

ONLY USE THE FOLLOWING TESTED AND APPROVED SOLUTIONS

MULTIPURPOSE SOLUTIONS

- Menicon Unique pH (Menicon)
- Boston[®] Simplus[®] (Bausch & Lomb)

HYDROGEN PEROXIDE

- Clear Care[®] (Alcon)
- Clear Care[®] with HydraGlyde (Alcon)



YOU MUST NOT USE

ALCOHOL-BASED CLEANERS, including but not limited to

- Miraflow Extra Strength Daily Cleaner (including Miraflow copies and equivalents)

ABRASIVE CLEANERS, including but not limited to

- Boston Advance (Bausch & Lomb)



TAP WATER

You should never rinse or rub the lenses in tap water. Aside from the well-known risk of infection, tap water contains no surfactants or lubricants to protect the surface from abrasion if the lens is rubbed between two fingers.

OTHERS, including but not limited to

- Boston One Step Liquid Enzymatic Cleaner (Bausch & Lomb)
- Optimum by Lobob[®] ESC Extra Strength Cleaner (Lobob Laboratories)
- Progent (Menicon)
- Boston Lab Cleaner (Bausch & Lomb)

Contamac has produced the above list of contact lens care solutions that are compatible with the Tangible Hydra-PEGTM coating on lenses made in the Optimum GP materials.

We advocate following the instructions indicated with each of the care solutions, including, where stated, rubbing and rinsing the lenses where appropriate.

You should never rub the lenses with a dry tissue or cloth. These are often very abrasive at a surface level and can damage the Tangible Hydra-PEG coating.

Lenses should not be stored dry, but always in a contact lens care solution, according to the instructions on the packaging of that solution.