

CONGRATULATIONS!

Your custom contact lens is coated with Tangible Hydra-PEG™ technology to enhance contact lens wettability, lubricity, and surface quality.

To maximize the life of your Tangible Hydra-PEG™ coating, it is very important to comply with the following Lens Care and Handling Instructions.

YOUR **BENEFITS** WITH TANGIBLE HYDRA-PEG



HEALTHY EYES

Fewer Deposits, Less Irritation
= Less discomfort



SURFACE PROPERTIES

Distinctly wettable and very lubricious



BETTER WEARING EXPERIENCE

Patients prefer their custom lenses coated in Tangible Hydra-PEG™



JUST HOW COMFORTABLE?

[Click](#) or scan to tell us how comfortable your new lenses are

LENS CARE & **HANDLING INSTRUCTIONS**



HANDLE WITH CAUTION, LENSES ARE SLIPPERY

Plug your drain to avoid losing your lenses.



DO NOT USE TAP WATER

Tap water poses a risk to your eyes and can damage the surface coating of your lens. Use fresh, sterile saline solution to rinse your lenses.

For more information visit o-thp.com/nowater



USE ONLY APPROVED TANGIBLE HYDRA-PEG™ CLEANERS

Menicon Unique pH Multipurpose Solution	by Menicon
Boston Simplus®	by Bausch & Lomb
Clear Care®	by Alcon
Clear Care® Plus with HydraGlyde	by Alcon

Replace solutions as per the pack instructions.
Never store your lenses dry.



WARNING!

Do not use **tap water**, **abrasive** or **alcohol-based** cleaners. These may damage or destroy the Tangible Hydra-PEG™ coating and should not be used.

Examples of products that should NOT be used:

Boston® Advance	by Bausch & Lomb
Miraflow (and equivalents)	by various companies
Progent	by Menicon
Optimum by Lobob ESC Extra Strength Cleaner	by Lobob Laboratories

Unsure about the solution you use? Check o-thp.com/solutions for further examples of approved and non-approved solutions. Always consult your eye care professional on the correct use of solutions, and how to rinse safely. **Say no to H2O.**