




Initial Lens Selection Guide

Select initial diagnostic lens based on corneal profile:

Flatter (Normal or Refractive Surgery) - 4000 sag

Average (Moderate Cones or Irregularities) - 4600-4800 sag

Steep (Highly irregular, steep cone, tilted or proud graft/host junction) - 5000 sag

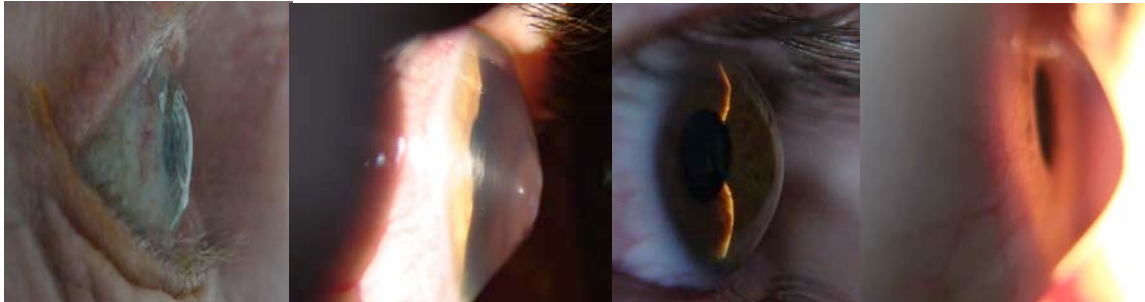

Onefit™ MED Simplified Fitting Regimen

Evaluating ideal clearance for optimal fit (microns)	Initial Application	30-45 Minutes	4+ hours
Central Zone/ Highest Elevation	250-300	200-225	150-175
Mid-Peripheral Zone	150-175	125-150	100-125
Limbal Zone	100-125	75-100	50-75
Edge Zone	Aligned to Sclera	Aligned to Sclera	Aligned to Sclera



blanchardlab.com

Need fitting help?

Call: 800-367-4009 or

Email: consultation@blanchardlab.com