

APPROVED SOLUTIONS LIST



English
Page 1



English
Page 2



English
Page 3



English
Page 4



French
Page 5



French
Page 6



German
Page 7



Italian
Page 8

Click language and region to jump to relevant page.

English

Contamac has produced this list of contact lens care solutions that are compatible with the Tangible Hydra-PEG™ coating on lenses made in the Optimum GP materials.

USE ONLY APPROVED TANGIBLE HYDRA-PEG™ CLEANERS

Multipurpose Solutions

Menicon Unique pH by Menicon
Boston Simplus® by Bausch & Lomb

Hydrogen Peroxide Solutions

Clear Care® by Alcon
Clear Care® Plus with HydraGlyde® by Alcon

DO NOT USE TAP WATER, ABRASIVE OR ALCOHOL-BASED CLEANERS

(These will damage or destroy the Tangible Hydra-PEG™ coating)

Abrasive Cleaners

Boston® Advance by Bausch & Lomb

or similar products

Alcohol Based Cleaners

Miraflow (and equivalents) by various companies

or similar products

Others

Progent by Menicon
Boston One Step Liquid Enzymatic Cleaner by Bausch & Lomb
Optimum by Lobob ESC Extra Strength Cleaner by Lobob Laboratories

or similar products

We advocate following the instructions indicated with each of the care solutions, including, where stated, rubbing and rinsing the lenses where appropriate.

You should never rub the lenses with a dry tissue or cloth. These are often very abrasive at a surface level and can damage the Tangible Hydra-PEG™ coating.

Lenses should not be stored dry, but always in a contact lens care solution, according to the instructions on the packaging of that solution.